


Sampler Removal


If the sampler requires replacement, it can be removed from the sensor module. Using a coin or fingers, remove the retaining screw from the sensor module.

- ▲ Also refer to package insert for sampler removal and installation updates.

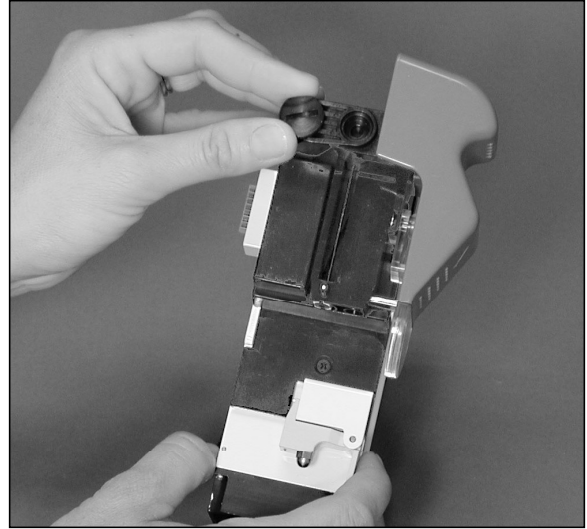

Remove the screw mount (includes spring).

Rotate the sampler into the capillary position, then pull it out of the sensor module.

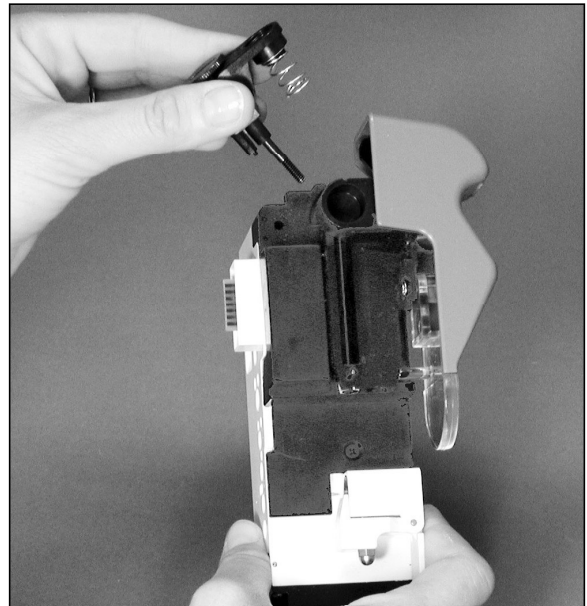

Sampler Installation

Install the new sampler (includes sample probe and wiper) in the sensor module. Insert the sampler alignment pin into the sensor module from a capillary mode orientation. Rotate the sampler into the closed position.

Install the screw mount (includes spring) into the sampler. Replace the screw and tighten.

- ▲ Do not overtighten the sampler screw.

